

RootDown™ Irrigation Syringe

A SIMPLE AND EFFICIENT METHOD FOR DELIVERING IRRIGATING SOLUTIONS

SINGLE | DUAL | TRIPLE



Shown with an ASI Micro Evacuator & Dual Irrigation System

CONVENIENT & ERGONOMIC

Ease of delivery is achieved through ergonomic design. The wand-style handpiece is held like a pencil, with the finger-friendly push button control located on the shaft. No thumb pressure is required. Access is simple - it fits standard handpiece holders, allowing placement on the holder bar alongside other instruments on the Advanced Endodontic System®.

FLEXIBILITY

Choose to deliver one, two, or three different solutions. The Irrigation syringe includes up to three color-coordinated irrigation handpieces with dedicated 250ml bottle reservoirs for each. The handpiece, tubing, and bottles are chemically resistant to common dental solutions, enabling delivery of irrigants of choice. Use with Max-i-Probe needles or compatible standard Luer type needles.

FINITE CONTROL

The irrigation system utilizes a flow control regulator, allowing the operator to change the rate of delivery from a slow to steady drip.

HOW IT WORKS

The titanium handpiece has a Luer connection that accepts Max-i-Probe or compatible Luer type irrigating needles. Simply place a compatible irrigation needle onto the Luer connection, fill the bottle reservoir with desired solution, flip the switch on the bottle bracket to on, and the system is ready to use.

A finger activated push button on the handpiece allows irrigating solution to flow. The solution reservoir uses air pressure to push the solution from the bottle. Push button control makes it possible to dispense without the constant thumb pressure required by hand held syringes.

EVACUATION & DRYING

Use ASI's Micro Evacuator for fast rinsing and drying. Just insert an Ultradent purple/blue tip onto the Micro Evacuator for quick evacuation of canal spaces. Titanium body and inert Luer type connector allows use with caustic irrigating solutions.

Note: Micro Evacuator requires connection to suction.

Ensure that a suction outlet is provided in your junction box connection when specifying office design plans with your architect. The active irrigation upgrade requires connection to the delivery system and is operated by foot control.

SPECIFICATIONS

Reservoir Capacity	250 mL per Bottle
Pressure Regulation Range	0-23 psi Max
Bracket System Dimensions	7" wide x 3.5" deep x 7.5" high
Handpiece & Valve Construction	Titanium
Bottle Construction	HDPE

MODELS AVAILABLE

90-2741S - Single Irrigation System

(Red color code designation)

90-2741D - Dual Irrigation System

(Red & green color code designations)

90-2741T - Triple Irrigation System

(as shown to right with Red, green & orange color code designations)



844.880.3636 | ASIDENTAL.COM | SALES@ASIDENTAL.COM

8811 American Way, Suite 130 | Englewood, CO 80112



ASI
Dental Specialties